

Purview ViVATM

**Share, view, store, and integrate
medical images**

Purview ViVA

Purview is a new generation imaging delivery platform enabling physicians easy and affordable access to medical images as an integrated part of their patient's complete health record.




Purview ViVA

- **Access:** extend access to referring physicians and patients
- **Sharing:** use a simple link to electronically share images and reports
- **Security:** secure HIPAA compliant communication channels and storage
- **Storage:** unlimited capacity that scales with your needs
- **Viewing:** view your images anytime anywhere on any device
- **Flexible:** avoid technology obsolescence



Step 1 - Connect

Purview will connect your modalities, existing PACS and viewing workstations via a secure connection to your ViVA™ cloud account.

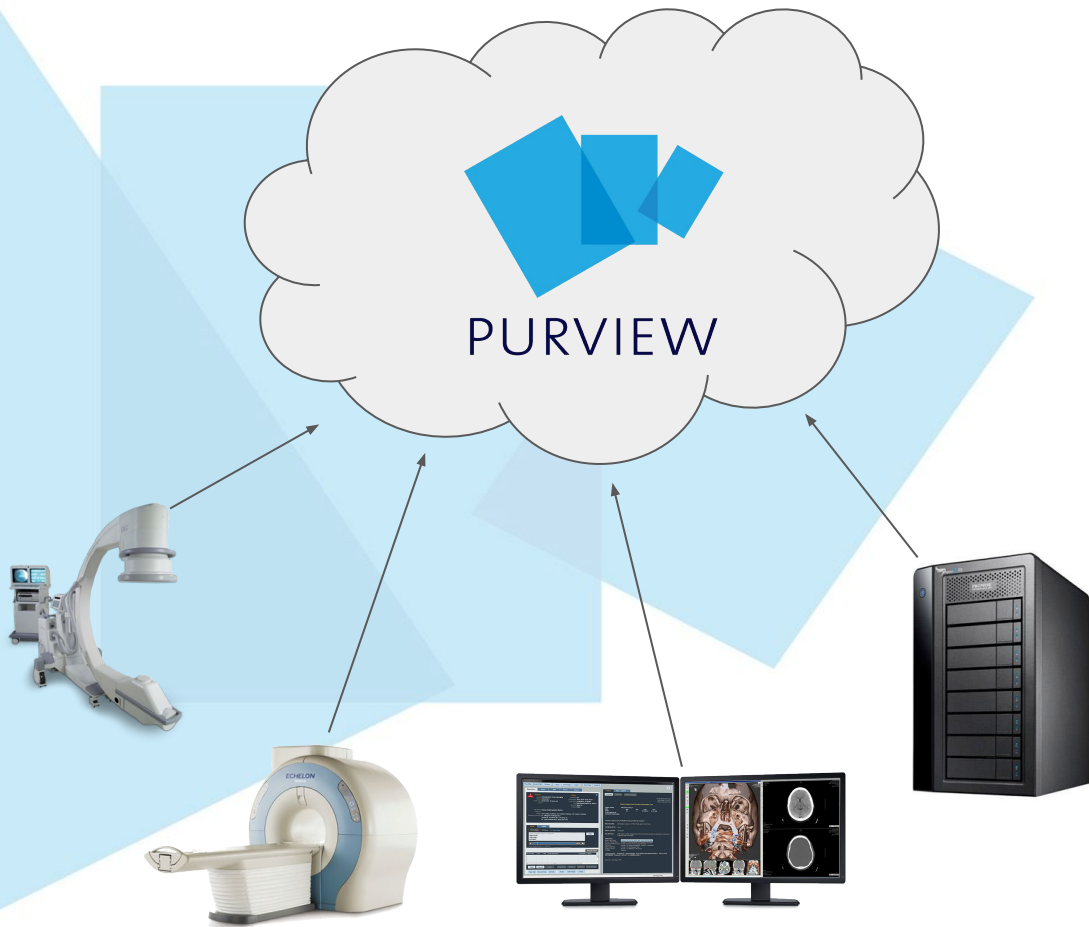

Step 2 - Access


View medical images on your existing viewing station, web browser, smartphone, or tablet.

Share and collaborate with physicians and patients.




Purview ViVA Log-In

Secure login with unique username and password.

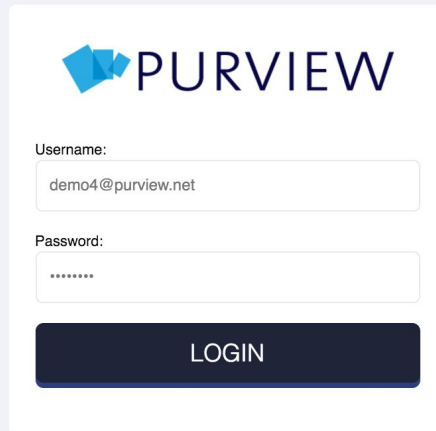

The login form is centered on a light gray background. It features the Purview logo at the top, which consists of a blue geometric icon followed by the word "PURVIEW" in a dark blue, sans-serif font. Below the logo, there are two input fields. The first is labeled "Username:" and contains the text "demo4@purview.net". The second is labeled "Password:" and contains a series of dots. Below these fields is a dark blue button with the word "LOGIN" in white, uppercase letters.

 **PURVIEW**

Username:

Password:

LOGIN


© Copyright 2016. All Rights Reserved.

Launching Purview ViVA

Access your cloud!

Search patient's first name, last name or patient ID.

With multiple patient studies, search within those studies as well.

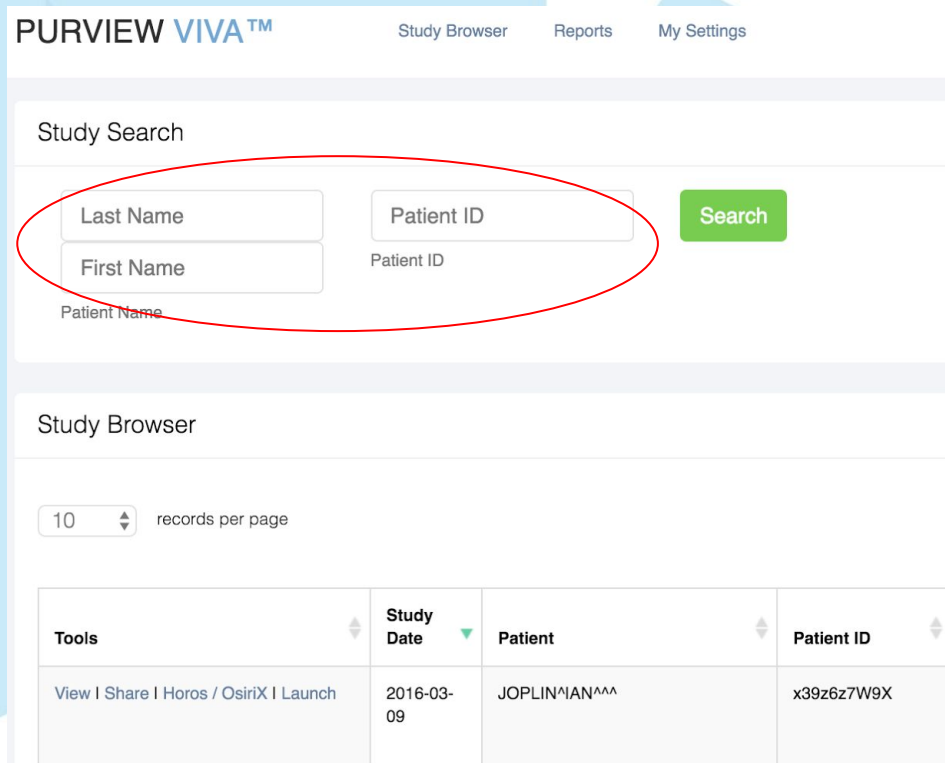

PURVIEW VIVA™

Study Browser Reports My Settings

Study Search

Last Name Patient ID

First Name Patient ID

Patient Name

Search

Study Browser

10 records per page

Tools	Study Date	Patient	Patient ID
View Share Horos OsiriX Launch	2016-03-09	JOPLIN^IAN^^^	x39z6z7W9X

Purview ViVA Tools

- **View:** view a study anytime, anywhere, on any device on our FDA approved universal viewer
- **Share:** share this study with another physician or patient
- **Download:** download a study to your viewing workstation
- **CD:** publish a CD

[View](#) | [Share](#) | [Horos / OsiriX](#) | [Launch](#) | [Download](#) | [CD](#)

Launching the Universal Viewer


Launch a universal viewer that will stream the images over the internet. No patient data is downloaded ensuring a secure and private view.

Study Browser		
10 records per page		
Tools	Study Date	Patient
View Share Horos / OsiriX Launch	2016-03-09	JOPLIN^IAN^^^

Universal Viewer

Our universal viewer provides powerful navigation and measurement tools.

- Window leveling
- Linear Measurement
- Compare current/prior studies
- 3D MIP/MPR
- 3D volume rendering

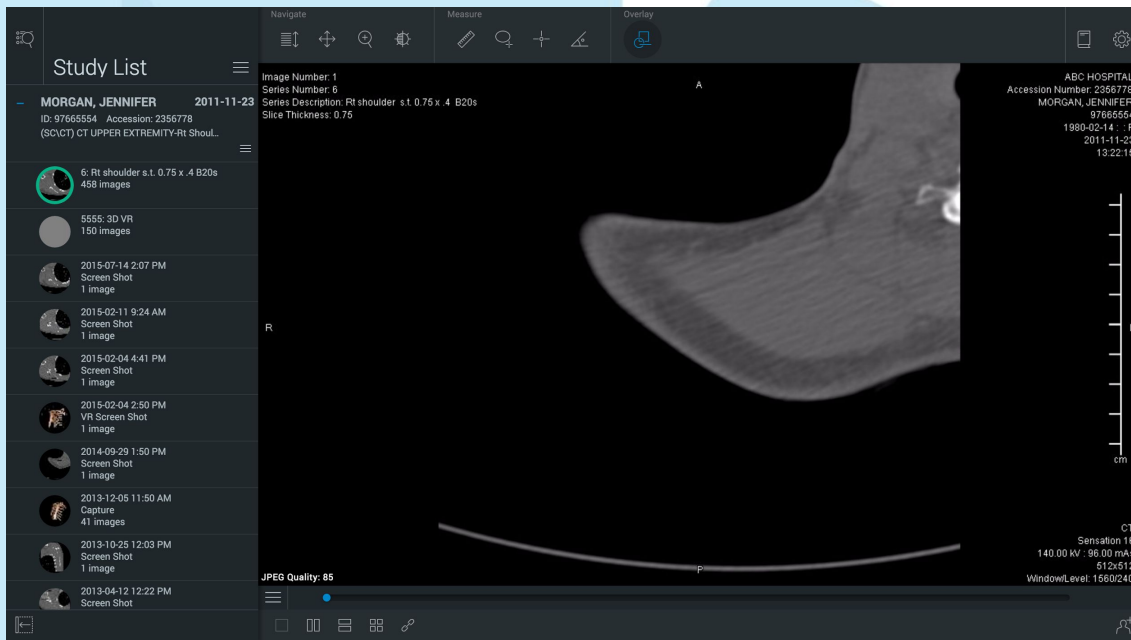

Powerful yet Simple Measurement & Navigation Tools

- Image Slice Scrolling
- Pan
- Zoom
- Window Leveling

- Linear measurement
- ROI measurement
- Point measurement
- Angle measurement

Navigate





Measure



Comparing Images


Compare current and prior images side-by-side and link them for easy evaluation.

Up to 4 series in the same viewer, including cine video playback.

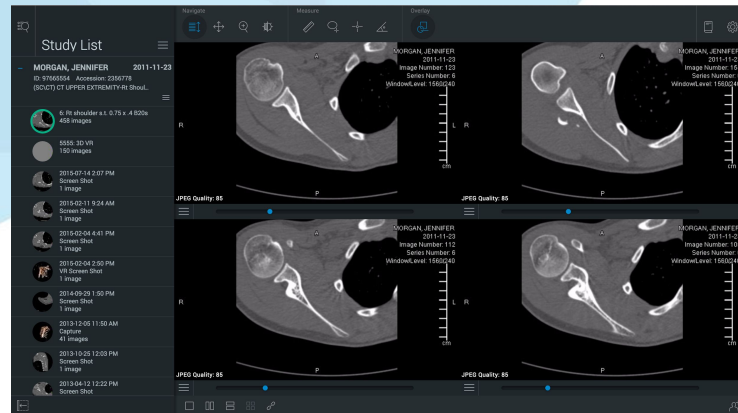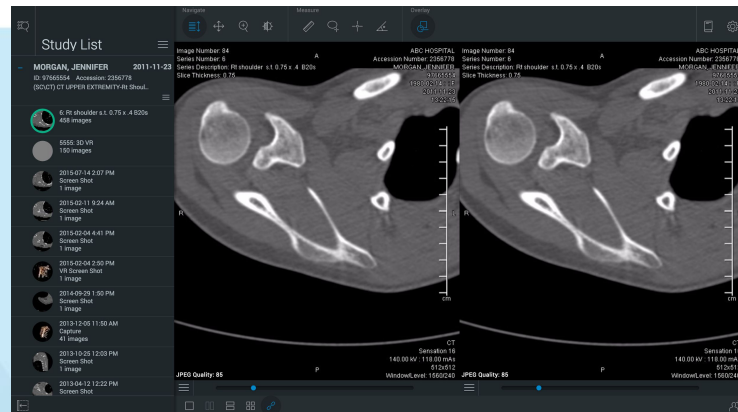


3D MIP/MPR

Navigate 3D studies
with multiplanar
reconstruction
(MPR).

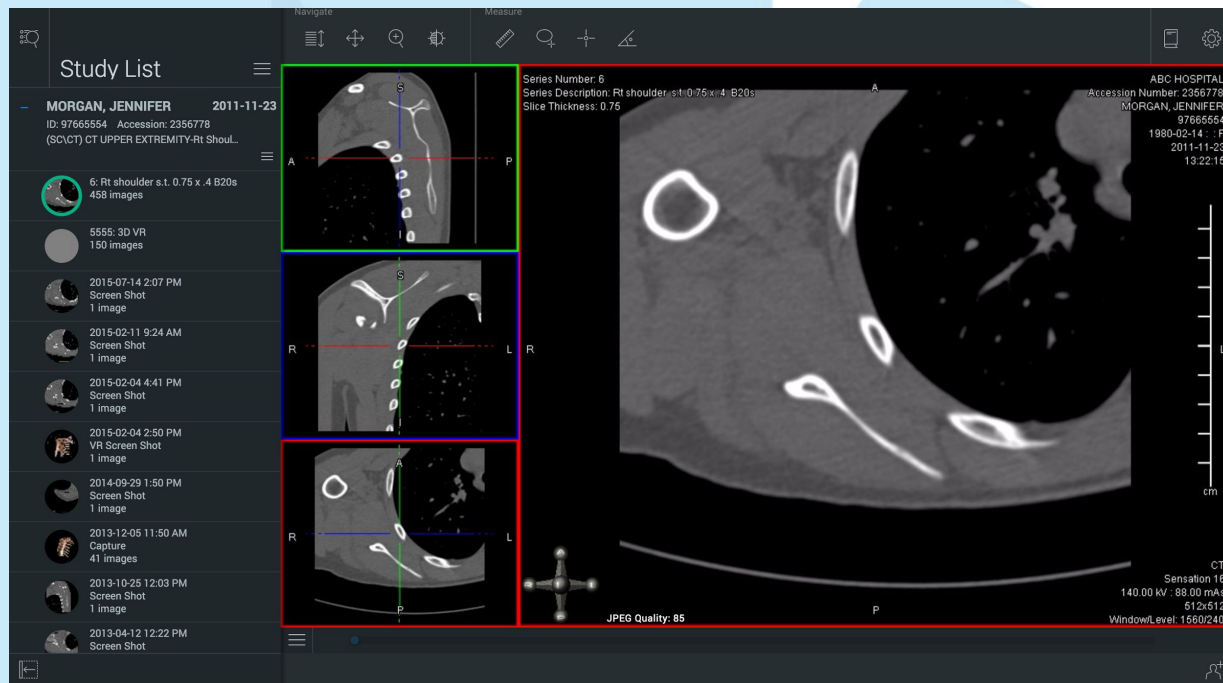


3D Volume Rendering

Perform 3D volume rendering and segmentation on any web-browser or mobile device.

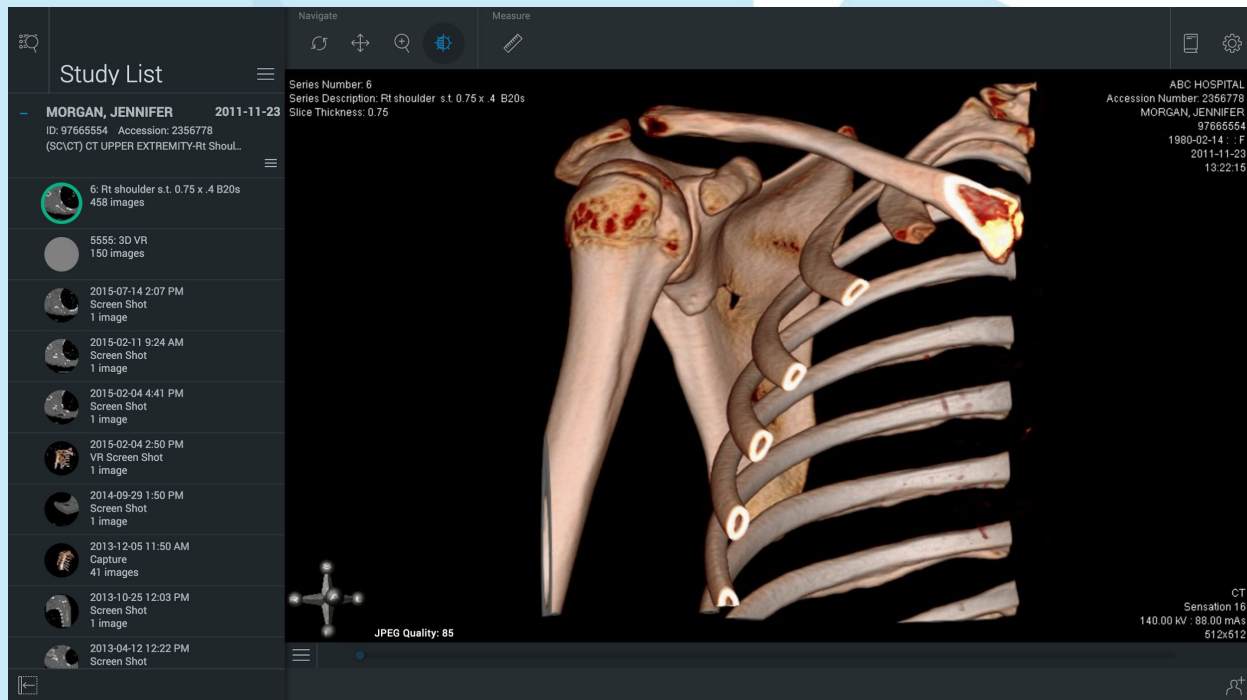

Collaboration in Real Time

Collaborate with other physicians in real time with simultaneous views

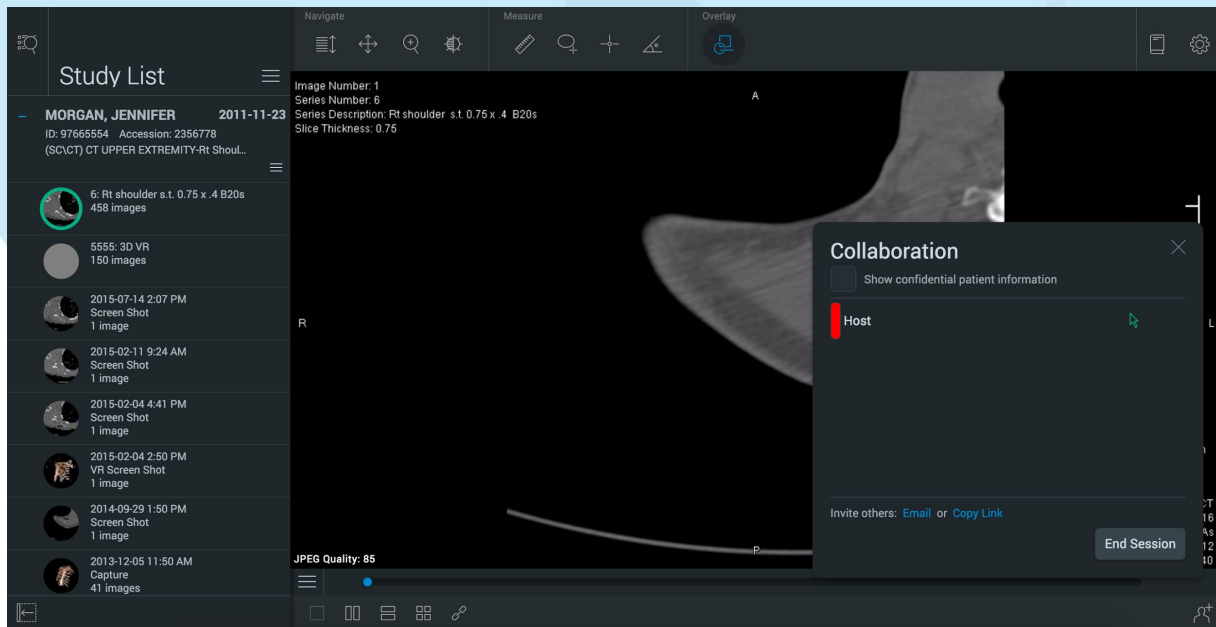

Step 3 - Share Medical Images

PURVIEW VIVA™ Study Browser Patient Browser Upload Reports My Settings demo3@securerad.com

Study Search

Last Name Patient ID Search

First Name Patient ID

Patient Name

Study Browser ⌵ ✕

10 ⌵ records per page Search within results:

Tools	Study Date	Patient	Patient ID	Modality	Study Description	Physician	Report
View Share History Launch Download CD	2011-11-23	MORGAN/JENNIFER^M^	97665554	SC:CT	CT UPPER EXTREMITY-RT SHO		New Mark as read Study Notes

Showing 1 to 1 of 1 entries ← Previous 1 Next →

Share with Referring Physicians

Create personalized login for your referring physicians to share patient images and reports.

Search within results: SMITH			
Modality	Study Description	Physician	Report
CR	+ PPD	SMITH^AAA	New Mark as read Study Notes
MR	C_spine^canine	SMITH^AAA	New Mark as read Study Notes
MR	RM CRANEO SIMPLE	SMITH^AAA	New Mark as read Study Notes

Share with Referring Physicians

Use secure URL links for temporary viewing or to download a study.

PURVIEW VIVA™

[Study Browser](#)[Patient Browser](#)[Upload](#)[Reports](#)[My Settings](#)

Share Images

To share viewing access for MORGAN^JENNIFER^^:

Share this URL

https://viva.purview.net/go.php?id=6MQY6MFY

Go Back

To share a download link for MORGAN^JENNIFER^^:

Share this URL

https://viva.purview.net/go_download.php?id=YWGXF7P7

Go Back

Patient Access

Provide instant web-based patient access to images and reports.

No need to burn CDs!

Purview™ gives you and your doctor secure and convenient online access to your medical images.

You control your medical images. View them, store them, share them with your doctors.

How?

In most cases the results of your imaging study will be sent directly to your doctor. Use this access code instead of a CD to access your study results and share with any of your doctors electronically.

1. Go to clinicname.purview.net
2. Click "I am a patient"
3. Enter your unique access code


Your Unique Access Code

EHR/EMR Integration

Connect images with patient data.

Purview creates a seamless connection between your EHR system and medical images.

Launch images directly from the patient record.

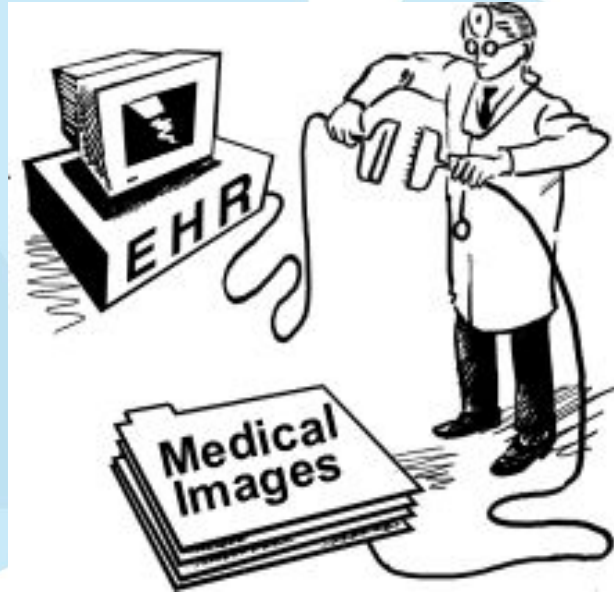

Horos Integration

Purview offers technical and operational support for Horos and provides a seamless connection with the Purview ViVA cloud.

Launch Horos and query your images directly from within Purview ViVA.





PURVIEWTM

Improving medical outcomes.

www.purview.net/contact-us

(800) 501-1537

experts@purview.net